


1 Assemble Sizing Chart First

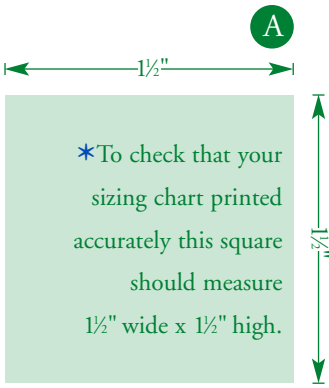

*To check that your sizing chart printed accurately this square should measure 1 1/2" wide x 1 1/2" high.

B Fold under on *Solid Fold Line* on page 2, align *Solid Lines* and arrows, and tape pages together.

2 Length Measurement

- A**
- Fold sizing chart on dotted line
 - Make a 90-degree angle
 - Place on hard surface
 - Put fold firmly against wall
 - Tape to prevent mis-sizing

use arrows to align pages

use arrows to align pages

Important
Before printing, turn off "fit to page" in your print options dialogue box.*

		8	adolescent/adult sizes
		7	
		6	
		5	
		4	
		3	youth sizes
		2	
		1	
		13	
		12	
		11	toddler sizes
		10	
		9	
		8	
		7	
		6	
		5	
		4	

children to adult sizes

Solid Line

use
arrows
to align
pages

B

- Size AFO only
- Place heel of AFO on sizing chart against wall
- Make sure to include posterior stop

C

- Trace AFOs for accuracy
- Match longest point to number on right
- If you are between or slightly above line, round up

D

Necessary Adjustments for Circumference

For all AFOs, TCFlex, DAFOs, KAFOs, SMOs with:

- If you are between or slightly above line, round up
Increase 1 full size
- Articulating hinges, posterior stop and instep strapping
Increase 1.5
- Turbo boot, posterior stop, hinges, footplates, instep, forefoot and toe strapping:
Increase 2 sizes

Fold under on *Solid Fold Line*,
align *Solid Lines* and arrows,
and tape pages together.



Fold up on *Dotted Fold Line*.

KEEPING PACE
Children's Orthopedic Footwear

use
arrows
to align
pages

Please be sure to accurately measure AFOs using this sizing chart and follow steps A–D listed on left. For best results, please verify square in left hand corner measures 1½" wide x 1½" high. KeepingPace cannot guarantee 100% accuracy due to discrepancies in various printing capabilities.

Keeping Pace, Inc.

contact us:
footwear@keepingpace.com
(888) 526-0020
14 Fuller St.
Gloucester, MA 01930
USA

visit us on the web:
www.keepingpace.com